

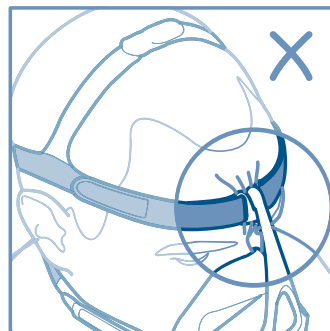
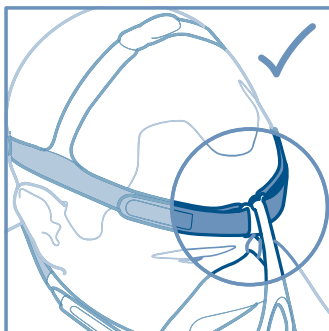
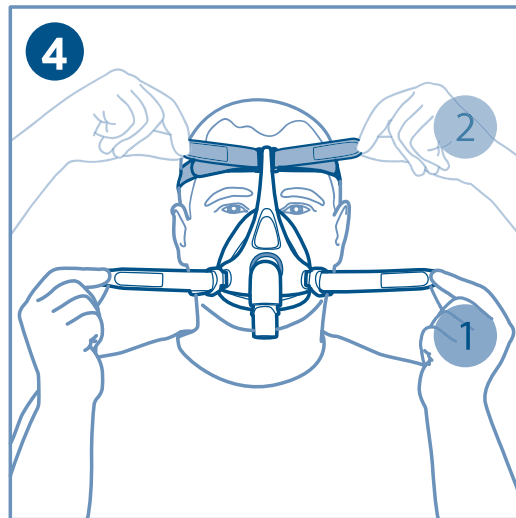


FITTING YOUR MASK

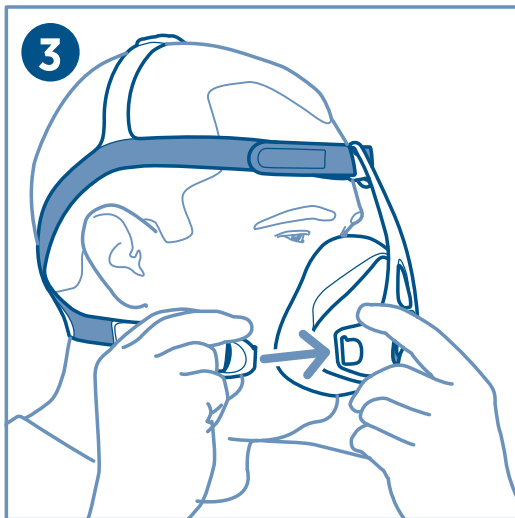

REF 185048495 REV B 2021-12 © 2021 Fisher & Paykel Healthcare Limited

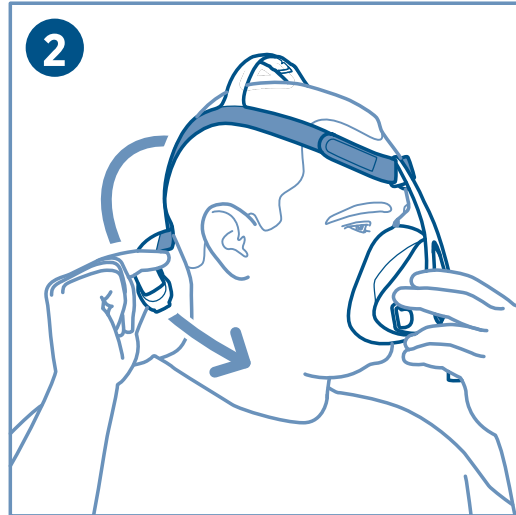

FULL FACE MASK






Use and Care Guide





**Thank you for choosing
your F&P Vitera.**

This Full Face mask has been designed to make it easier for you to fit CPAP therapy into your life. This is your mask Use and Care Guide. Please read this entire guide before you first use your mask. Following the steps in this guide, and referring to the images and diagrams will assist you with assembly, fitting and daily use of your F&P Vitera Full Face mask.

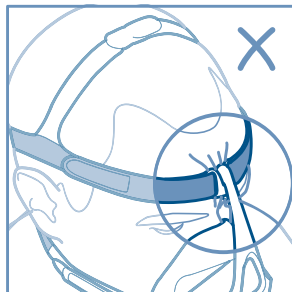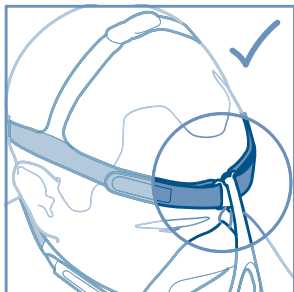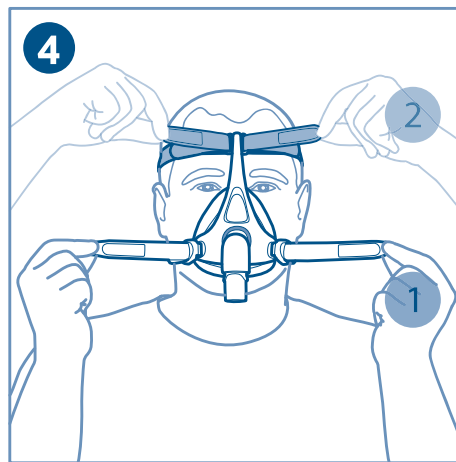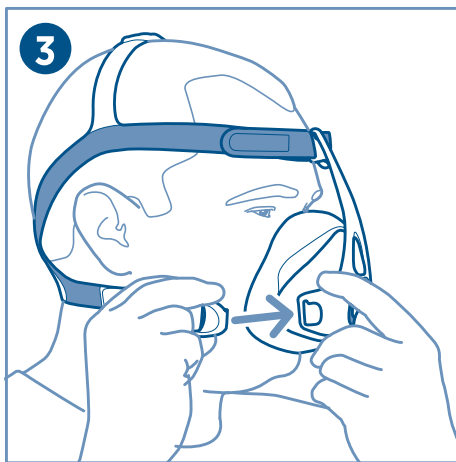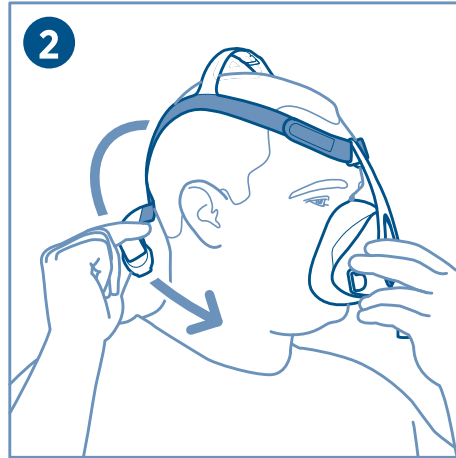

LANGUAGES

PAGE	LANGUAGE
1	English
8	Spanish (Español)
15	French - CA (Français)

Manufacturer 

Fisher & Paykel Healthcare Ltd,
15 Maurice Paykel Place,
East Tamaki, Auckland 2013

PO Box 14 348 Panmure,
Auckland 1741, New Zealand

Tel: +64 9 574 0100
Fax: +64 9 574 0158
Email: info@fphcare.co.nz
Web: www.fphcare.com

Importer/ Distributor

Australia (Sponsor)

Fisher & Paykel Healthcare Pty Ltd,
19-31 King Street, Nunawading,
Melbourne, Victoria 3131.

Tel: +61 3 9871 4900
Fax: +61 3 9871 4998

Austria

Tel: 0800 29 31 23
Fax: 0800 29 31 22

Benelux

Tel: +31 40 216 3555
Fax: +31 40 216 3554

Brazil

Fisher & Paykel do Brasil,
Rua Sampaio Viana, 277 cj 21,
Paraíso, 04004-000,
São Paulo – SP, Brazil

Tel: +55 11 2548 7002

China

代理人/售后服务机构:
费雪派克医疗保健(广州)有限公司, 广州
高新技术产业开发区科学城科丰路31号G12
栋301号

电话: +86 20 32053486
传真: +86 20 32052132

Denmark

Tel: +45 70 26 37 70
Fax: +46 83 66 310

Finland

Tel: +358 9 251 66 123
Fax: +46 83 66 310

France 

Fisher & Paykel Healthcare SAS, 10 Av. du
Québec, Bât F5, BP 512, Villebon-sur-Yvette,
91946 Courtaboeuf Cedex, France

Tel: +33 1 6446 5201
Fax: +33 1 6446 5221
Email: c.s@fphcare.fr

Germany

Fisher & Paykel Healthcare GmbH,
Deutschland, Österreich, Schweiz,
Wiesenstrasse 49,
D 73614 Schorndorf, Germany

Tel: +49 7181 98599 0
Fax: +49 7181 98599 66

Hong Kong

Tel: +852 2116 0032
Fax: +852 2116 0085

India

Tel: +91 80 2309 6400

Ireland

Tel: 1800 409 011
Fax: +44 1628 626 146

Italy

Tel: +39 06 7839 2939
Fax: +39 06 7814 7709

Japan

Tel: +81 3 5117 7110
Fax: +81 3 5117 7115

Korea

Tel: +82 2 6205 6900
Fax: +82 2 6309 6901

Mexico

Tel: +52 55 9130 1626

Norway

Tel: +47 21 60 13 53
Fax: +47 22 99 60 10

Poland

Tel: 00800 49 119 77
Fax: 00800 49 119 78

Russia

Tel. and Fax: +7 495 782 21 50

Spain

Tel: +34 902 013 346
Fax: +34 902 013 379

Sweden

Tel: +46 8 564 76 680
Fax: +46 8 36 63 10

Switzerland

Tel: 0800 83 47 63
Fax: 0800 83 47 54

Taiwan

Tel: +886 2 8751 1739
Fax: +886 2 8751 5625

Turkey

İthalatçı Firma: Fisher Paykel Sağlık Ürünleri
Ticaret Limited Şirketi,
İletişim Bilgileri: Ostim Mahallesi 1249.
Cadde No:6, Yenimahalle, Ankara, Türkiye
06374,

Tel: +90 312 354 34 12
Fax: +90 312 354 31 01

UK

Fisher & Paykel Healthcare Ltd,
Unit 16, Cordwallis Park, Clivemont Road,
Maidenhead, Berkshire SL6 7BU, UK

Tel: 0800 132 189
Fax: +44 1628 626 146

USA/Canada

Tel: 1800 446 3908
or +1 949 453 4000
Fax: +1 949 453 4001

Rx only  0123